

# Got the Point?



## CCPoint®

*World's first point-of-care test  
for Rheumatoid Arthritis*

- *Early diagnosis – early treatment*
- *Reliable – correct diagnosis of Rheumatoid Arthritis*
- *Rapid – 10 minutes to results*

EURO-DIAGNOSTICA



# Quick Reference Instruction CCPoint® Test

Before performing the test; Read and adhere strictly to the provided Instructions For Use



Figure 1: Clean test site with the pre-injection swab (not provided in the kit)



Figure 2: Place the lancet on the test site (lancet is not provided in the kit)



Figure 3: Push down the lancet to activate



Figure 4: Collect the blood sample with the provided disposable pipette



Figure 5: Transfer the blood sample into the sample port and wait 30 seconds



Figure 6: Lift the lid of the test device




Figure 7: Add 5 drops of buffer into the buffer port and allow to run for 10 min

T	C	Positive: 2 lines appear (C and T)
T	C	Negative: 1 line appears (C)
T	C	Invalid: 1 line appears (T)
T	C	Invalid: no lines appear

Figure 8: Interpretation of the results



Clavis.com 2009-06

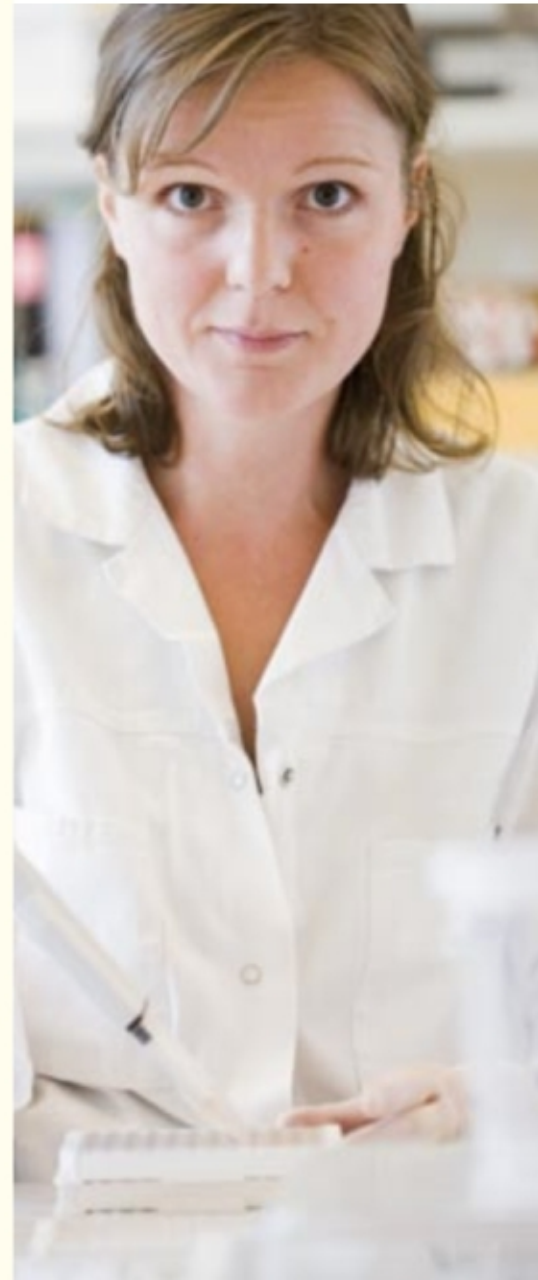

CCPoint 20

Point-of-care test for qualitative determination of anti-CCP (20 devices)

EURO-DIAGNOSTICA

